

Dynacast® Prelude

Synthetic Splint System



- **Versatile splinting system for individual patient application**
- **Clean modern alternative to plaster of Paris slabbing**
- **Strong and light with no plaster mess**
- **All-in-one roll is easy and quick to apply**

Dynacast® Prelude

Synthetic Splint System

- Stronger and lighter than traditional plaster of Paris slabbing
- Quick setting for early immobilisation
- High strength for excellent durability
- Roll format allows versatility of splints
- Soft and mouldable for excellent comfort

Dynacast® Prelude is a synthetic splinting system specifically incorporating fibreglass, covered by a polypropylene padding, to offer easy application for the support and immobilisation of fresh fractures and soft tissue injuries. It is presented in a roll format for versatility in use and application.



• Strong, Light & Durable

The strength of the resin coated fibreglass of Dynacast® Prelude ensures that the injury site is fully supported at all times, and the splint is resistant to breakdown.

• Quick Setting and Early Weight Bearing

Dynacast® Prelude has an initial set of 3-5 minutes and reaches weight bearing strength in 20 minutes.

• No Mess or Waste

The continuous roll format ensures the clinician will only use what is required and provides versatility to choose the length of splint to suit the indication. With no plaster mess Dynacast® Prelude is ideally suited to the theatre and ward environment.

• Comfortable for the Patient

Soft polypropylene padding covers the fibreglass splint which minimises the risk of pressure points and naturally aids moisture transmission away from the skin to enhance patient comfort. The custom mouldable nature of the splint ensures a perfect patient fit.

• Indications

Immobilisation of fresh fractures and support of soft tissue injuries and rheumatoid arthritis. Can be used for both acute and rehabilitation treatment phases with Dynacast® Prelude strap kits.



Ordering Information

Dynacast® Prelude

ORDER CODE	DESCRIPTION	PACK
71418-00	2.5cm x 4.6m	1 x 1 roll
71420-00	5cm x 4.6m	1 x 1 roll
71421-00	7.5cm x 4.6m	1 x 1 roll
71415-00	10cm x 4.6m	1 x 1 roll
71416-00	12.5cm x 4.6m	1 x 1 roll
71417-00	15cm x 4.6m	1 x 1 roll
71419-00	20cm x 2.3m	1 x 1 roll

Dynacast® Prelude Solo

ORDER CODE	DESCRIPTION	PACK
72963-00	5cm x 25cm	1 x 10
72963-01	7.5cm x 30cm	1 x 10
72963-02	7.5cm x 87cm	1 x 10
72963-03	10cm x 38cm	1 x 10
72963-04	10cm x 75cm	1 x 10
72963-05	12.5cm x 75cm	1 x 10
72963-06	15cm x 30cm	1 x 10

Dynacast® Prelude Strap kits

ORDER CODE	DESCRIPTION	PACK
71415-18	Upper Limb Kit = 3 red straps	1 x 10 kit
71415-19	Lower Limb Kit = 3 blue straps	1 x 10 kit



BSN medical Ltd, PO Box 258, Willerby, Hull HU10 6WT
Tel: 01482 670100 Fax: 01482 670111
Email: advice.uk@bsnmedical.com www.bsnmedical.co.uk

® Registered Trademark
LB/PRELS/00223/0306
© BSN medical Ltd. 2007